

## PD Camphor Mono-Chlorophenol (CMCP)

---

### Solution for root-canal disinfection

#### Composition

p-Chlorophenol 35%, Camphor 65%

#### Properties

PD Camphor Mono-Chlorophenol, also known as CMCP, is an active disinfectant for the treatment of infected root-canals & periapical infections.

#### Indication

- Disinfection after pulpectomy
- Treatment of post-traumatic inflammation
- Pulp dressing

#### Direction for use

- Dry the root-canal thoroughly
- Following a pulpectomy: insert 1 or 2 drops of preparation into the root-canal & allow few minutes for it to take effect. Remove any excess solution with a cotton pellet or an absorbent paper point
- Following the extraction of gangrenous pulp: use as a dressing by inserting into the root-canal with a smooth broach & obturate temporarily. Leave 'in situ' for at least 3 days. Make sure the canal is completely disinfected after the period. If this is not the case, repeat the dressing before the final obturation

#### Storage

Store at room temperature between 5°C & 30°C, protected from direct light.

#### Packaging

Bottle of 15ml

