

PD Pengha Iodoform Paste (Alvogyl)

Antiseptic, analgesic & haemostatics alveolar paste

Composition

Penghawar Djambi 4%, Iodoform 16%, Ethylaminobenzoate 26%, Eugenol 14% , Oil of Mint 8%, excipient ad 100%

Properties

PD Pengha Iodoform Paste combines 4 active substances of long-lasting therapeutic activity. Penghawa-Djambi is a haemostatic, Iodoform is an antiseptic & Ethyl Aminobenzoate is a local anaesthetic which acts quickly & remains effective for a long time. Eugenol strengthens the analgesic & antiseptic effects of the other components.

Indications

- Post-extraction sockets
- Alveolar osteitis
- Prevention of post-extraction alveolar complications

Directions for use

As a non-resorbable dressing:

-Apply the paste on a gauze which can be removed after 2-6 days OR

Apply directly into the socket:

-As the paste is not resorbable, do not suture after application. The paste remains active for several hours. The patient should be advised not to swallow the paste

Precautions

When closing the jar, make sure the lid is tightly screwed on.

Once the jar has been opened, the paste may darken after exposure to air but darkening does not alter the effectiveness.

Packaging

Jar of 15g

