

PULPOTEC

BARBOLEO

For the treatment of
vital molars


Pulpotomy with PULPOTEC® is **simple**, **quick** and **efficient**

Thanks to PULPOTEC®
the technique of pulpotomy is now optimized
with minimum discomfort for the patient

- > Pulpitis in vital molars can be treated by pulpotomy with no time wasted
- > Traditional operating techniques without the risk of extrusion through the apex
- > No costly or sophisticated equipment needed

A long-lasting treatment
which preserves the tissular structure of the tooth

Directions for use : see overleaf

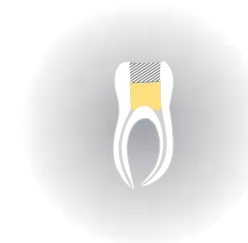

Recommendations for first-time users of
PULPOTEC®

1. Keep strictly to all the indications supplied in the packaging leaflet.
2. Treat pulpitis only on **vital** molars, both permanent and deciduous. (The sole exception to this golden rule is the treatment of infected deciduous teeth, even if they present an abscess).
3. This treatment is to be used exclusively for **molars**.
4. **Pulpotomy** is simple but best results are obtained when it is carried out scrupulously.
A clean pulpotomy is best obtained with high-speed rotary instruments which avoid tearing the radicular filaments and efficiently remove **all** the cameral pulp.
5. A little residual blood in the pulpal chamber will not affect the success of the pulpotomy.
6. Never use Pulpotec® when performing a pulpectomy.

Catalogue N° 6
Page N° 37

PRODUITS DENTAIRES SA



N° 70144 N° M10041



Rue des Bosquets 18
Case postale
CH-1800 Vevey, Switzerland

Telephone + 4121 921 26 31
Telefax + 4121 921 3979
produits.dentaires@bluewin.ch
Website: www.pdsa.ch