

## PD Root Canal Enlarger – EDTA Gel

---

### **Gel for the chemical widening of the root canal**

#### **Composition**

Edetate tetrasodium (EDTA) 190mg, Cetyltrimethylammonium Bromide 10mg, excipient ad 1ml

#### **Properties**

PD EDTA Root Canal Enlarger is a bactericidal & non-toxic gel to be used for softening the dentinal walls of the root canal. It acts by chelation of the Calcium ions & eliminating of inorganic debris. This process facilitates the subsequent mechanical widening of the root canal during treatment.

#### **Indications**

- Chemical widening of the root canal

#### **Directions for use**

- Introduce the preparation directly into the root canal
  - for the gel, use the specially adapted tip of the syringe
- Drill the canal mechanically in the usual manner & re-apply the preparation as often as necessary
- Rinse thoroughly with water

#### **Packaging**

Syringe of 3g