

## PD Tricresol & Formalin (FormoCresol)

---

### Solution for root-canal disinfection

#### Composition

Tricresol 35%, Formaldehyde(40%)19%, excipient ad 100%

#### Properties

- Formocresol, thanks to its antibacterial activity, had proved as efficient in vital pulpotomy as for root-canal treatment.
- The Formaldehyde acts as a powerful, long-lasting germicide & also renders inert any remaining organic materials.
- The Cresol acts as an efficient antiseptic & also reduces any irritation that could be caused by the Formaldehyde.

#### Indications

- Treatment of 4th degree caries
- Root-canal disinfectant dressing, to be used specially when fistula have developed

#### Directions for use

- Clean & prepare the canal. Dip cotton gauze into the solution, squeeze until no longer dripping & insert into the canal
- Seal with a Temporary Cement
- Remove the temporary filling & the gauze after 3 to 5 days & check the asepis of the canal. Where necessary, the treatment can be renewed
- Close cavity with a permanent filling

#### Storage

Store at room temperature between 5°C & 30°C, protected from direct light.

#### Packaging

Bottle of 15ml

